# iX Series Service Setup Guide

[iX Series Service Setup Guide](#)

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
**SleepWell App Introduction**

**SleepWell enables patients to self-assess therapy effectiveness upon waking up**

Please confirm that your service provider has sent the invitation to your personal email. If you didn't receive the invitation letter, please contact your service provider or physician.

SleepWell App


1. Self-monitor your treatment progress and outcomes.
2. Access to feedback and resources from physicians.
3. Treatment education center provides guidance and troubleshooting information.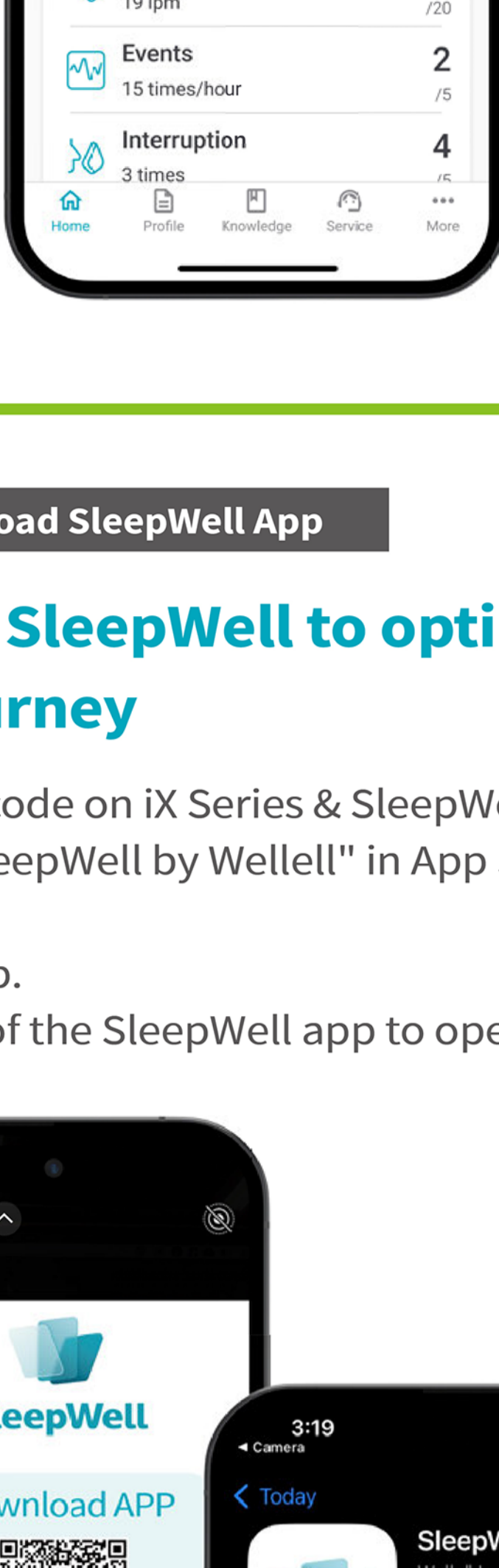

**Steps to Download SleepWell App**

**Download SleepWell to optimize your iX journey**


1. Scan the QR code on iX Series & SleepWell Starter Guide. Or search "SleepWell by Wellell" in App Store or Google Play.
2. Install the app.
3. Tap the icon of the SleepWell app to open the app.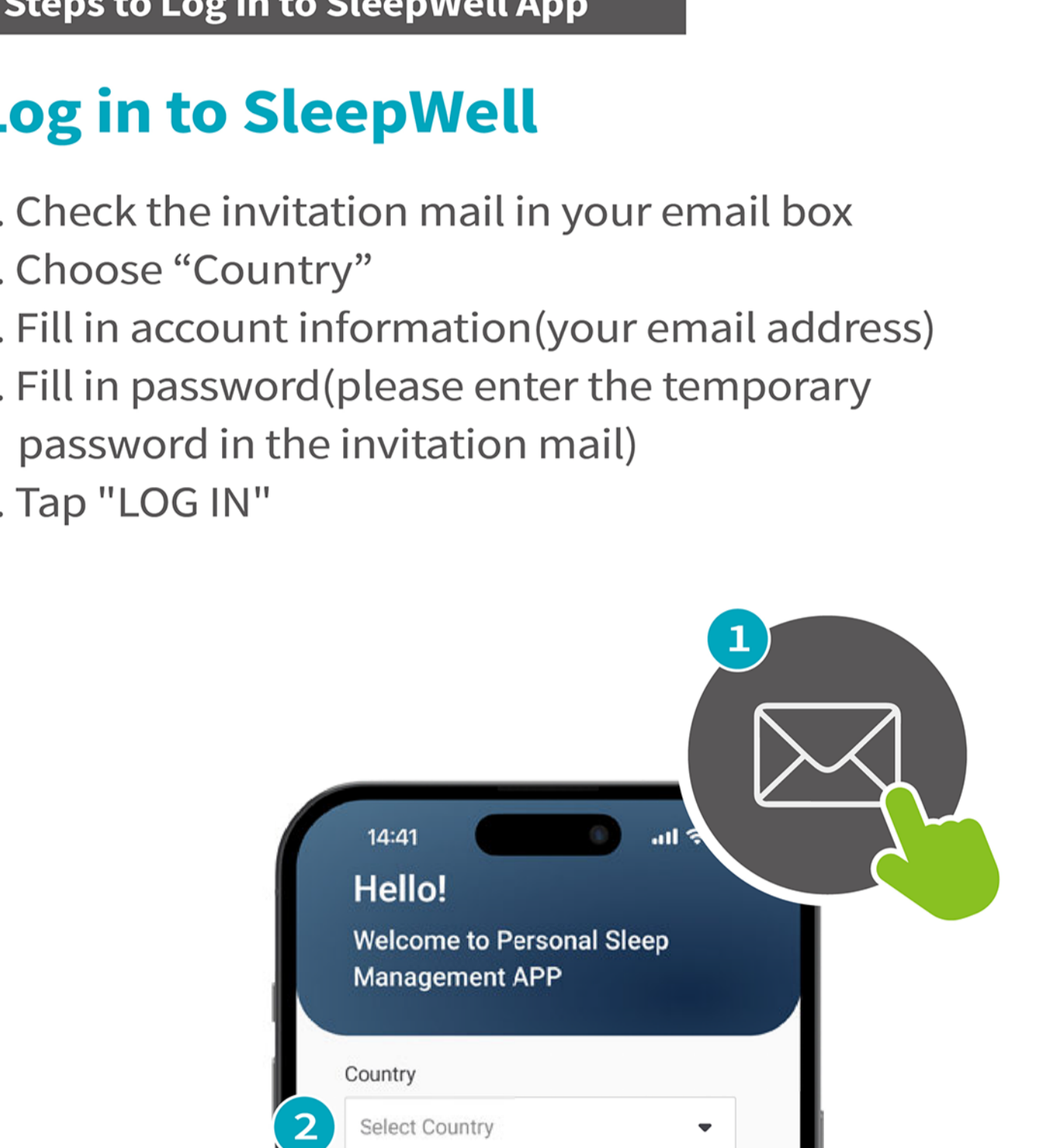

**Steps to Log in to SleepWell App**

**Log in to SleepWell**

1. Check the invitation mail in your email box
2. Choose "Country"
3. Fill in account information(your email address)
4. Fill in password(please enter the temporary password in the invitation mail)
5. Tap "LOG IN"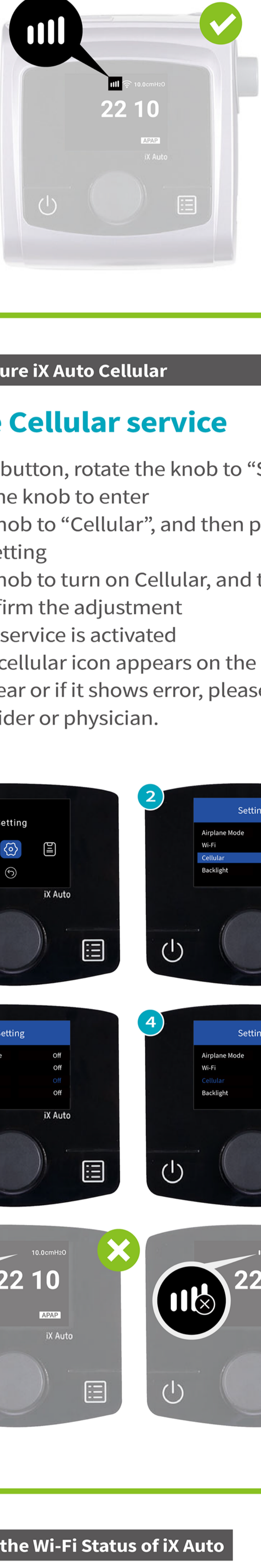


If the log in process fails, please contact your service provider or physician

**How to Check the Cellular Status of iX Auto**

**Upload treatment data via built-in Cellular**


1. Check the Cellular icon on the screen
2. If the icon appears, the Cellular signal is activated
3. If the icon doesn't appear, please follow the steps to configure the Cellular service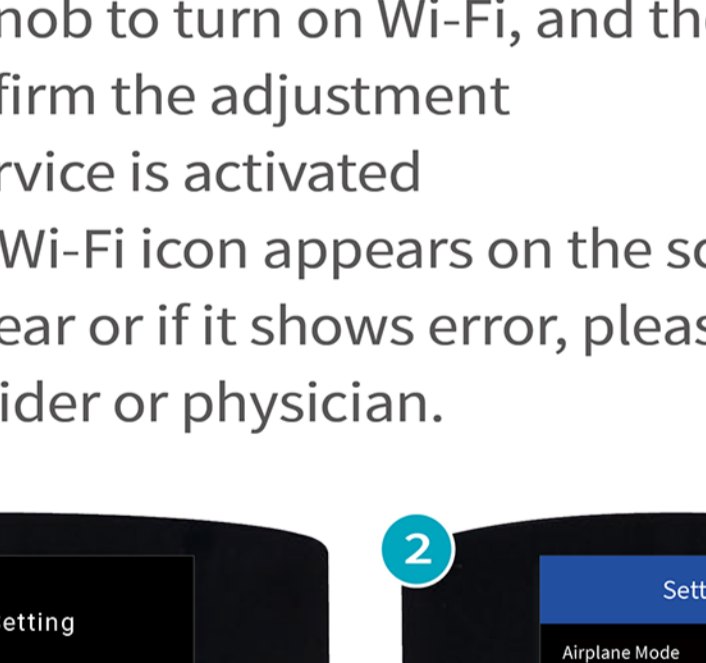

**How to Configure iX Auto Cellular**

**Configure Cellular service**


1. Press MENU button, rotate the knob to "Setting", and then press the knob to enter
2. Rotate the knob to "Cellular", and then press the knob to adjust the setting
3. Rotate the knob to turn on Cellular, and then press the knob to confirm the adjustment
4. The Cellular service is activated
5. Check if the cellular or if it shows error, please contact your service provider or physician.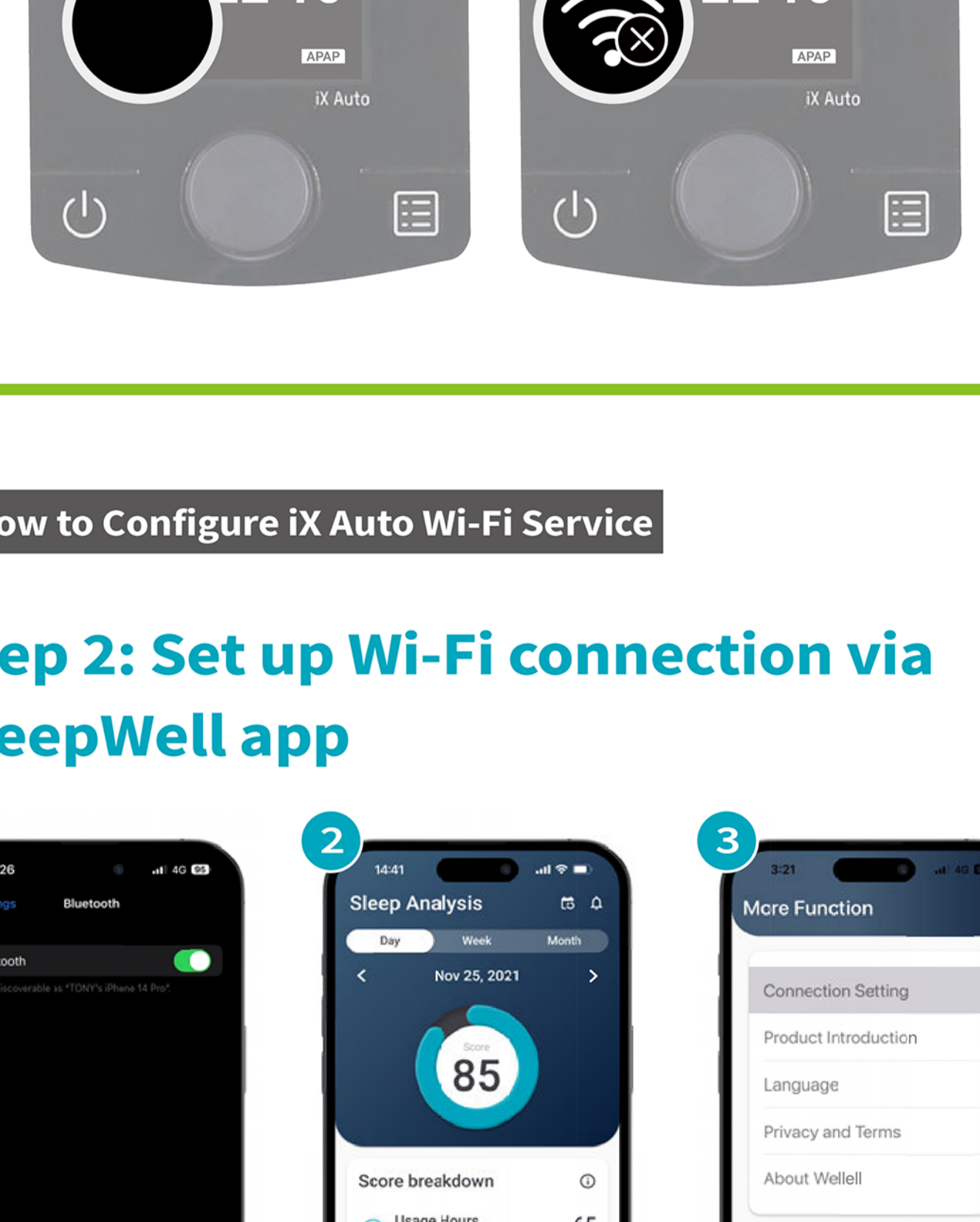

**How to Check the Wi-Fi Status of iX Auto**

**Upload treatment data via Wi-Fi**

1. Check the Wi-Fi icon on the screen
2. If the icon appears, the Wi-Fi signal is activated
3. If the icon doesn't appear, please follow the steps to configure the Wi-Fi service

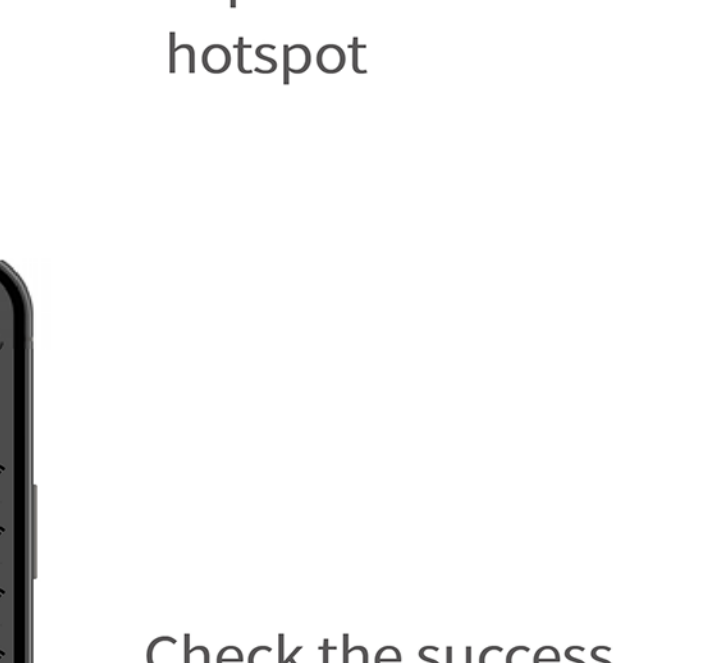**How to Configure iX Auto Wi-Fi Service**

**Step 1: Turn on Wi-Fi service on iX Auto**

1. Press MENU button, rotate the knob to "Setting", and then press the knob to enter
2. Rotate the knob to "Wi-Fi", and then press the knob to adjust the setting
3. Rotate the knob to turn on Wi-Fi, and then press the knob to confirm the adjustment
4. The Wi-Fi service is activated
5. Check if the Wi-Fi icon appears on the screen. If it doesn't appear or if it shows error, please contact your service provider or physician.

**Step 2: Set up Wi-Fi connection via SleepWell app**

1. Turn on the Bluetooth on mobile phone
2. Enter "SleepWell", tap "More"
3. Tap "Connection setting"

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